


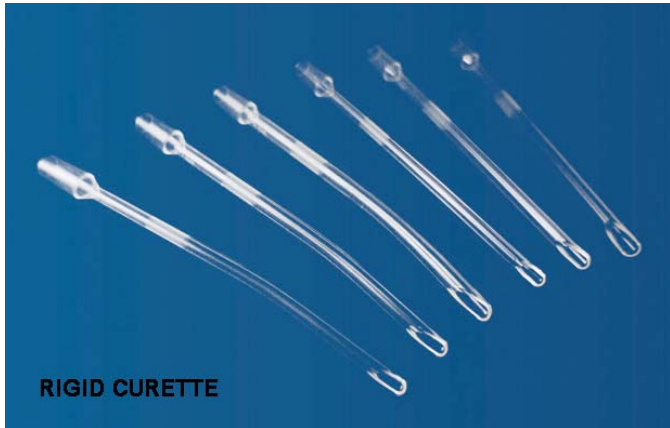

Rigid Curette



Panpac Medical Corporation
6F-2, No. 202, Sec. 3, Ta-Tong Road,
Shi-Chih District, New Taipei City,
Taiwan

TEL : 886-2-86472242

FAX : 886-2-86472770

LATEX
FREE



STERILE R



0434

EC REP

Wellkang Ltd (www.CE-marking.eu)
29 Harley St., London W1G 9QR, UK

The Panpac Curette is designed and manufactured with care in our laboratories, in order to reduce potential patient trauma. Panpac curette combine a minimum outer diameter with a maximum inner diameter for fast, safe and total evacuation by suction of uterine cavity.

INSTRUCTIONS FOR USE

Measure uterine depth. Dilate the cervix if necessary. Introduce INTRACURETTE through the cervical canal into the uterus up to the fundus. Connect the curette to the suction system. Operate by rotating the curette 360° to aspirate throughout the uterus cavity accurately. Repeat the operation two or three times. Withdraw INTRACURETTE from the uterus while continuing the rotating movement. Dispose of after use.

INTENDED USE

Used for uterine evacuation and curettage in voluntary pregnancy termination or following miscarriage.

SPECIFICATION

Rigid Straight		Rigid Curved	
Product No.	Sizes	Product No.	Sizes
CuRs06	6mm	CuRc06	6mm
CuRs07	7mm	CuRc07	7mm
CuRs08	8mm	CuRc08	8mm
CuRs09	9mm	CuRc09	9mm
CuRs10	10mm	CuRc10	10mm
CuRs11	11mm	CuRc11	11mm
CuRs12	12mm	CuRc12	12mm

Single patient use only.

PRODUCTION

Manufactured by press injection, the curette are automatically transferred into environmentally controlled rooms. Where they are inspected and shaped as required. All curette are checked visually. This inspection is necessary to eliminate any flaw. The curettes are then "flamed", to round off sharp edges. In the final stage, the curette are individually packaged before being sterilized.

CAUTION

Do not move the curette sideways. Do not go beyond the depth measured by the hystrometer. Read the instructions before using the device. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose of after use according to the laws in force. For use only under trained medical staff.

U.S. Federal law restricts this device to sale by or on the order of a physician.

Do not reuse for avoiding user may be infected

PRINCIPLE:

The diameter of the curette to be used corresponds to the number of weeks of pregnancy (e.g. an 8mm curette should be used for an 8 week pregnancy).