



**Flexible Curette(or Cannula)**

Panpac Flexible curette(or cannula) is a flexible plastic curette incorporates with vacuum aspiration syringes to provide a safe, efficient and fast vacuum aspiration method to evacuate the contents of the uterus.

### SPECIFICATION

Product No.(SKU)	Sizes	Color of Remarks
KC04	4mm	Yellow
KC05	5mm	Green
KC06	6mm	Blue
KC07	7mm	Black
KC08	8mm	Red
KC09	9mm	White
KC10	10mm	Tangerine
KC12	12mm	Purple

### INDICATIONS

Use of Panpac Flexible Curette(or Cannula) with the suction system for uterine evacuation and curettage in voluntary pregnancy termination or following miscarriage.

### CAUTION

1. Do not move the curette sideways.
2. Do not go beyond the depth measured by the hystrometer.
3. Read the instructions before using the device.
4. Use only if the package is intact.
5. Dispose after single use.
6. Use only by or under the supervision of a trained physician.



Panpac Medical Corporation  
6F-2, No. 202, Sec. 3, Ta-Tong Road,  
Shi-Chih District, New Taipei City,  
Taiwan

TEL : 886-2-86472242

FAX : 886-2-86472770



Wellkang Ltd ([www.CE-marking.eu](http://www.CE-marking.eu))  
29 Harley St., London W1G 9QR, UK

### DIRECTION FOR USE

1. Measure uterine depth.
2. Dilate the cervix if necessary.
3. Introduce the approximately size of Flexible Curette through the cervical canal into the uterus up to the fundus.
4. Connect the Flexible Curette to the suction system.
5. Operate the procedure by rotating the curette 360° to aspirate throughout the uterus cavity accurately. Repeat the operation two or three times.
6. Withdraw the curette from the uterus while continuing the rotating movement.

### PRODUCTION

Each Flexible Curette is sterile and packaged individually. The curette should be discarded after a single use.

### PRINCIPLE:

The diameter of the curette to be used corresponds to the number of weeks of pregnancy

Approximate Uterine Size (weeks LMP*)	Approximate Cannula Size
5-7 LMP	5mm
7-9 LMP	6mm
9-12 LMP	7-12mm

\* LMP = Last Menstrual Period